## **Subject: ACCU-CHEK Inform and Base revised cleaning procedure**


Based on customer feedback it has been determined that there is a potential for the ACCU-CHEK Inform meter and base unit to overheat, causing melting and smoking when the meter/base connectors and base unit circuitry are exposed to excessive amounts of disinfectant or cleaning solution.

Or

Our internal tests have confirmed that exposure of the meter/base connectors and base unit circuitry to excessive amounts of disinfectant or cleaning solution, and not drying the meter and base unit thoroughly prior to charging, can result in overheating, melting, and smoking while the meter is docked for charging.

To prevent this issue from occurring with the ACCU-CHEK Inform meter and base unit, Roche Diagnostics have reviewed the cleaning procedure and recommendation. Please make sure to follow these new guidelines.

We are also applying a sticker on the base unit between the words ACCU-CHEK Inform and the Roche logo that states “Ensure meter/base (including connectors) are thoroughly dried after cleaning or disinfecting”.

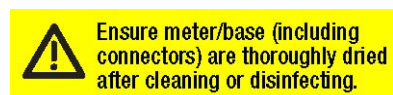The sticker simply serves as a reminder about the drying step after cleaning or disinfecting. We are inclosing spare stickers so you can apply them on the current base at your facility.

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